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Dust extractor Set

Contents

1	Scope of delivery.	13
	1.1 Scope of delivery for complete set	13
	1.2 Scope of delivery for re-fitting set.....	13
2	Safety instructions.....	13
3	Installation.....	14
	3.1 Installation Dust Removal Unit	14
	3.2 Installation Saw Blade.....	15
4	Cleaning and maintenance	15
5	Accessories/Spare parts	16
6	Accessories / Optional	17
7	Warranty	18
8	Environmental protection	18
9	Manufacturer and customer service address	18

Dust Extractor Set

1 Scope of delivery

1.1 Scope of delivery for complete set

Vaccum Cleaner
ECO nonwoven filter
Nonwoven filter bag (1 pc.)
Tube complete
Hose clamp
Complete with saw TITANIUM II oder ECO II
Saw Blade
Dust Removal Unit
Tools
Operating manual

1.2 Scope of delivery for re-fitting set

Vaccum Cleaner
ECO nonwoven filter
Nonwoven filter bag (1 pc.)
Tube complete
Hose clamp
Saw Blade
Dust Removal Unit
Tools
Operating manual

On request, we can alter this standard configuration.

Please note the wording on your invoice.

For more information, see the chapter Accessories

2 Safety instructions

All safety instructions contained in this operating manual are marked with **!**.


Read the operating manual and the locally applicable occupational safety regulations and ensure that these are observed before using the vacuum cleaner for the first time. Please ensure that this operating manual is kept in a safe place for later use and enclose it when selling or otherwise passing on the vacuum cleaner to a third party.

- ! Intended purpose: This device is intended to be used for the extraction of fine, dry grinding dust.
- ! When vacuuming, the filter may never be removed.
- ! Do not rivet or screw any signs or symbols onto the device. This can render the protective insulation ineffective. We recommend using adhesive signs.
- ! Only ever use undamaged plugs and cables. Check the plug and cable at regular intervals.
- ! The mains voltage and voltage specification given on the device must be in agreement.
- ! Only use original accessories.
- ! Wear personal protective gear, goggles, ear protectors and gloves
- ! The device may not be operated in potentially explosive atmospheres

3 Installation


3.1 Installation Dust Removal Unit

- Pos.1 to clip on with Pos.3 and to tighten with Pos. 2.
- Pos. 5 (o- ring) to put over Pos. 4 and to grease bajonett fitting.
- Pos. 1 to connect the bajonett fitting over Pos. 4.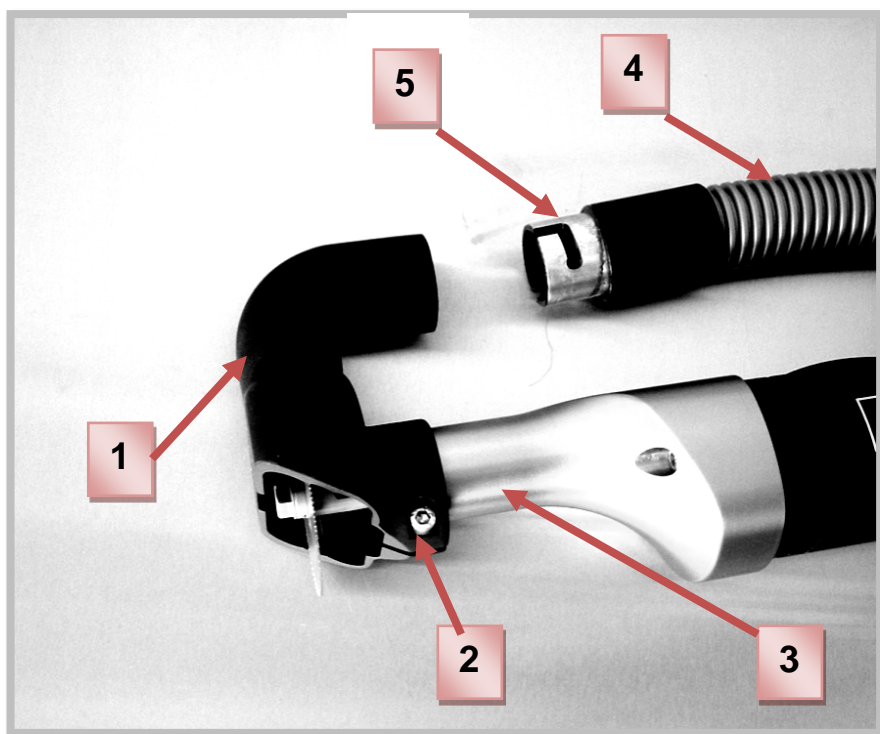

3.2 Automatic mode

- To draft the saw blade and manual the screw to edge upwards.
- To apply open- ended wrench on shaft to secure.
- To tighten the screw with socket wrench.

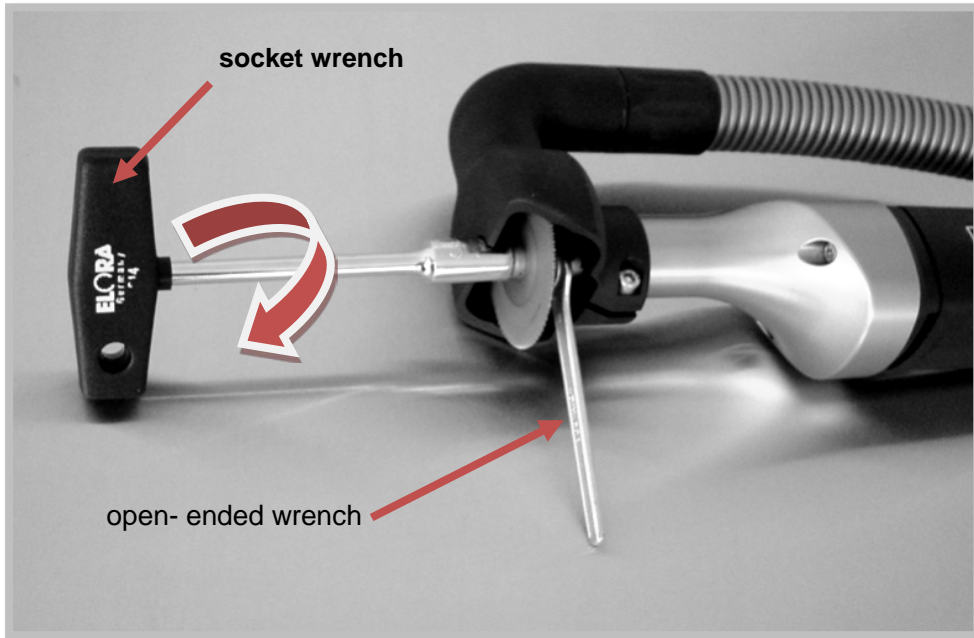Always at the change from the saw blade, apply both wrenches! Socket wrench to attract and open-ended wrench to secure the screw!

4 Cleaning and maintenance

Before performing any cleaning or maintenance work, disconnect the mains plug.


Repairs may only be performed by approved customer service centres or by specialist personnel who are familiar with all relevant safety regulations.

Ensure that all safety checks stipulated by the relevant locally applicable regulations for mobile commercially used devices are observed.

!	Bajonett fitting between dust removal unit and vacuum tube must be greased with special lubrication grease.
!	The bajonett fitting should be spaced to get cleaned and greased.
!	The o-ring should be spaced to be under examination to form cracks and deformation.

- Exchange the motor protection filter every three months (otherwise suction loss or damage to the suction turbine can occur).
- Exchange the exhaust filter once a year.

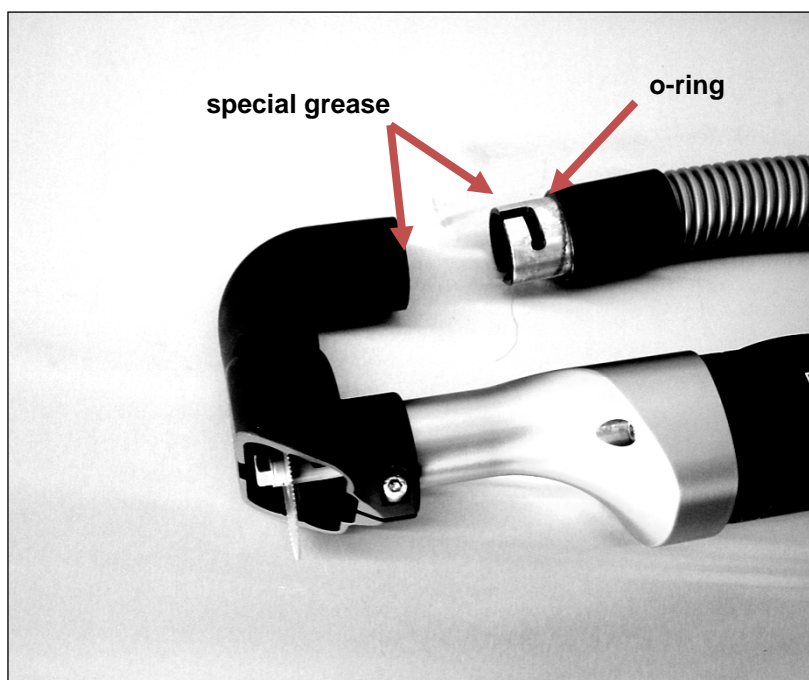5 Accessories / Spare parts

Tube complete with 2m

Hose clamp

Socket wrench with T-handel 11 mm

Tube with special grease

Open- ended wrench 11 mm

Spare screw for saw blade

Fixation ring for screw

6 Accessories / Optional

Hanging kit with handle, height adjustable
(left figure for all saws except ECO II and right figure
for ECO II)

Magnetic shelf for saw blades and tools

7 Warranty

We offer a warranty on the Dust Extractor in compliance with statutory and country-specific regulations (presentation of invoice or delivery note required). The warranty period is at least 18 months. In the event of material or production defects, we provide repair of the device free of charge. Any attempts by users to carry out their own repairs will void any warranty claims.

Damage caused as a result of incorrect treatment, overload or normal wear and tear are not covered by the warranty. Please contact our after sales service department.

In the event of a warranty claim or repair, please enclose a copy of the invoice or delivery note with the device

8 Environmental protection

Ensure that packaging, end of life devices and accessories are sent for environmentally friendly recycling. Electrical appliances must not be disposed of with normal household waste, but sent to the responsible local disposal and recycling centre. Ask your specialist dealer for more detailed information.

9 Manufacturer and after-sales service address

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ORTHO-MEDICAL GmbH